

Eyelids & Eyelashes Cleanser with tea tree

EYE CLEAN Lid Cleaner & **Cleandex**

Unique formulation for eliminating
Demodex Mite by tea tree oil extract

USE FOR DRY EYE, BLEPHARITIS, STYES, MEIBOMIAN GLAND DYSFUNCTION, TO INCREASE CONTACT LENS COMFORT, AFTER MAKE-UP REMOVAL, AND BEFORE AND AFTER EYE SURGERY

- ✓ **Alcohol Free**
- ✓ **No Sulfates**
- ✓ **Fragrance Free**
- ✓ **Preservative Free**
- ✓ **No Parabens**



* Beautifully designed package with
30 individually wrapped pre-moistened
lid wipes (2.0ml, 0.07fl.oz / 30 wipes)



* EYE CleanDex Foam
(50ml, 1.69FL.OZ , 120 100% Cotton Pad)

INDICATIONS of EYE CLEAN LID CLEANER



Dry Eye



Red Itchy Eyelids



Demodex



Blepharitis



Contact Lens
Discomfort



After make-up
Removal



Stye



Before and After
Cataract Surgery



Cleaning
Eyelids & Eyelashes


Directions : General Cleansing of Eye Lids and Lashes.

1. Prior to treating to eye area ensure that your hands are clean
2. Open the sealed package and remove the wipe
3. Make sure to use a separate wipe for each eye area
4. Remove all mucous secretions and residues from the eye lids and lashes by wiping in downward directions. Make sure to also clean between the eyelashes and in the corner of your eyelids
5. Do not re-use.

About EYE CLEAN LID & LASH Cleaner with tea tree.

[SOOTHES DRY EYES, BLEPHARITIS, DEMODEX, AND ITCHY, INFLAMED SKIN CONDITIONS]

USE FOR DRY EYE, BLEPHARITIS, STYES, MEIBOMIAN GLAND DYSFUNCTION, TO INCREASE CONTACT LENS COMFORT, AFTER MAKE-UP REMOVAL, AND BEFORE AND AFTER EYE SURGERY

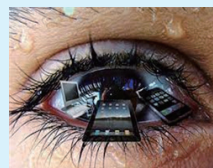

☒ **Dry eye
condition**



☒ **Kill
Demodex mites**

The EYE CLAEEN LID & LASH Cleaner will never irritate your delicate eye lid and lash.
EASY EYE LID APPLICATION INSTRUCTIONS AND FAST ONSET OF ACTION

NO NEED TO RINSE OR REMOVE!

**For the best results, we recommend to use the product
twice a day : in the morning and evening**

 Manufacture by **BIO Optics Co., Ltd.**

#1003 Sicox Tower, 484 DeunChondaero, Jungwonku, Sunghamsi, Kyungkido, 13229, Korea

www.biooptics.co.kr | www.리드클리너.kr

 **MADE IN KOREA**

T. +82-31-747-4850 | F. +82-31-747-4820 | E. eyemed@daum.net