

주식회사 더엘스타

THELSTAR Co., Ltd.

2020.08.19













Has become a necessity for life

A disposable mask




FACT CHECK

A disposable mask that is commonly used.
What makes you uncomfortable?





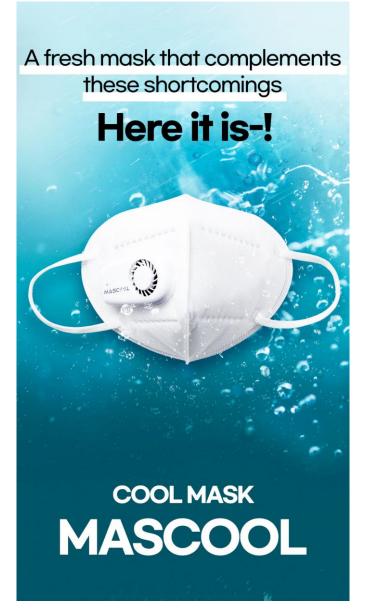





FACT CHECK 100/2



Q. You said it is a cool mask, is it easy to breathe?

Mascool is simply a refreshing mask with a ventilator.





Mounting the external air backflow prevention pad through the ring on the backside of MASCOOL. (Silicon pad detachable.)

Ventilator releases carbon dioxide, humidity, and heat inside the mask by air circulation, make you easier to breathe and keep a pleasant state inside the mask and also no fogging on your glasses.



Q. I'm wearing glasses. Just wear a mask, I can't see the front. Isn't it damp?

Mascool effectively releases humidity and heat. Would you like to see the photos taken with the thermal imaging camera below?



















CE certification





Invention patent

Design Registration







MASCOOL exclusive model Ryu Jikwang



It was introduced to the media.







Source: Light up your daily llife




How to Mount the Fixing Ring



- Prepare the MASCOOL device and the fixing ring.
- Fit the inner bump of the fixing ring to the bump rear of the MASCOOL.
- Rotate the fixing ring counterclockwise ($\triangle \rightarrow \bigcirc$) to lock it.
- Rotate the fixing ring clockwise ($\bigcirc \rightarrow \blacktriangle$) to release it.

How to Exchange Silicon pad, Damper

* Additional purchase available for individual parts

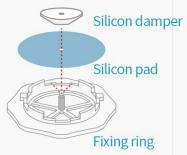

- Insert the fixing ring silicon pads - silicon damper in order.
- The silicon damper fits the convex part downwards.

※ It will allow comfortableness by removing the silicon pads and dampers on days with adequate air quality.

Charging Light Guide

Press	Long Press	Once	Twice	3 Times
Mode	ON/OFF	Low	Middle	High
Color	LIGHT ON / OFF	Blue	Purple	Red

About 90 minutes to fully charged / Red light off at fully charged

Capable of charging with C-type cable





PACKAGE CONTENTS

X C-type charger is not included.

MASCOOL M (Includes 1 fixing ring): B / W











Just like the mask you've been using,






Simply punch a hole in any mask without a cutting mat. Air valve, mascool, and circulator can be attached.







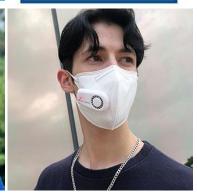



Cutting tips: Turn the cutting handle while pressing the cutting body support!







Fine dust mask

KF, KN, public mask



2 holes are also available

Breathe twice as comfortable



Cotton mask


for cold weather, fashion



Any position possible

Wherever I want

How to use mascut

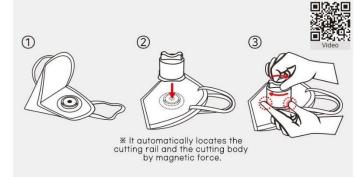

- 1 Place the cutting rail where you want to cut the hole.
- 2 After stacking the mask, put on the cutting body.
- 3 While pressing the cutting body support firmly, turn the cutting handle clockwise 1.5 ~ 2 turns.

%Precautions

Please read the precautions carefully!

- After use, assemble safely and store it.
- Keep out of reach of infants.
- Be careful when using the product as there is a risk of injury to your hands.
- Excessive impact on the body may cause product damage.
- Be careful not to cut your hand on the cutting blade when separating the product.
- The cutting blade is sharp, so be careful not to lean toward your face or body.
- If any foreign substances remain on the cutting part of the product, clean it with a cotton swab or hair dryer.
- Be careful not to get your hands caught in the cutting guard or cutting rail when assembling because of its magnetic properties.
- If you throw away the cutting edge, there is a risk a being cut, so please wrap it in thick paper.